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## 6 vaginal silicone dilators, a delicate dilation aid

### Indications

For vaginal complaints and penetrative sex problems due to the following:

- vaginismus,
- dyspareunia ( painful intercourse),
- as a result of gynaecological surgery,
- following birth.

### Instructions for use

Prior to use, wash the dilators with hot water and a mild soap. Use the dilators in a place that is comfortable and where you will not be disturbed and make sure that you won't be interrupted by the telephone or anything else while you are using the dilators. Place the dilators ready in order of size. The recommended position for using the dilators is to lie flat on your back, with your knees bent and legs slightly spread apart. You may, however, also use the dilators stood up with one leg placed on a chair. When inserting, it is advisable to use a lubricant such as SYLK or even water to lubricate the tip of the dilator or the entrance to the vagina. Breathe calmly and gently insert the smallest dilator into the vagina as far as is comfortable. Leave the dilator in the vagina or entrance to the vagina for approx. 8-10 mins, then insert and remove the dilator without fully taking the dilator out of the vagina. Repeat this movement approx 20 times.

You may find that when you begin using the dilators that you can only insert the tip of the dilator into your vagina, this is totally normal. Unless otherwise instructed by your doctor, you should repeat this procedure 3 times a week for 2 weeks. Once you are able to insert the smallest dilator without feeling any pain, we recommend immediately switching over to the next size up. For this stage as well as for each step up to the next size, it is necessary to insert and leave the previously used dilator in the vagina for 4-5 mins. Then remove this dilator and insert the next larger size and leave it in the vagina for approx. 8-10 mins. Then insert and remove the dilator without fully taking the dilator out of the vagina. Repeat this movement approx 20 times. You should repeat this procedure 3 times a week for 2 weeks. This also applies when swapping to the other sizes. As soon as you are able to insert the largest dilator painlessly, you may then consider penetrative sex.

### Contents:

- 6 silicone dilators

Color	Diameter A/B inch	Length C inch
Yellow	0.43/0.51	2.56 inch
Red	0.67/0.75	2.95 inch
Blue	0.75/0.87	3.54 inch
Purple	0.87/0.98	4.33 inch
Dusky pink	1.14/1.30	5.12 inch
Grey-blue	1.38/1.61	5.91 inch

### Care instructions:

Always make sure that the dilators are dry before you put them back in the storage bag. The dilator set may be dipped into boiling water in the event of infection and/or for cleaning purposes.

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Kessel, Marketing & Vertrieb GmbH